

Gas Lift Stools

Hamiltomedical
built for life

CUSTOMER INSTRUCTIONS

PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE USING THIS PRODUCT.
If you have trouble understanding these instructions contact your dealer or Hamilton customer support, (888) 871-5600 before attempting to use this product; otherwise injury may occur.

Hamilton Medical assumes no responsibility for damage or injury caused by improper assembly, installation, use, or maintenance of these products.



1. READ AND FOLLOW ALL DIRECTIONS.
2. NEVER place stool on a cart, table, stand or other objects; Stool could fall and INJURY MAY OCCUR.
3. DO NOT allow children to play with stool; Stool has moving parts and INJURY MAY OCCUR.
4. Periodically, recheck tightness of all nuts, bolts and screws.
5. Stool must ALWAYS be positioned on level surface.
6. NEVER stand or ride on stool; Stool not to be used for user transport; INJURY MAY OCCUR.
7. DO NOT use stool in or with ANY type of vehicle or trailer. Stools have not been tested or approved for use by an occupant in any type of vehicle or trailer.
8. Immediately remove from service; any stool with broken or missing parts, torn vinyl, or other visible damage.
9. USE ONLY AUTHORIZED REPLACEMENT PARTS.
10. NEVER EXCEED the recommended **weight capacity of 250 lbs.** (113.4 kg).
11. SAVE THESE INSTRUCTIONS for future reference and training.

Stool Weight Capacity = 250 lbs. (113.4 kg.)

Base Assembly

1. Place the **Upholstered Seat** on a clean, flat surface, cushion side down.
2. Push the stem of the **Gas Cylinder** into the **Stool Swivel Plate** (pre-attached to the Upholstered Seat).
3. Slide the **Foot Ring (optional)** over the **Gas Cylinder**, so that the **Foot Ring** faces the bottom of the **Upholstered Seat**.
4. Place the **Stool Base** over the end of the **Gas Cylinder** and gently tap it down.
5. Turn the entire assembly upright on the floor
6. If you have the optional **Foot Ring**, tighten the **Foot Ring Adjusting Knob** now.

Back Assembly

7. Slide the steel **Slotted Back Strap** into the **Stool Swivel Plate**.
8. Insert the **Back (depth) Adjusting Knob** to lock the **Slotted Back Strap** into place.
9. Hold the **Upholstered Back** up to the "seat side" of the **Slotted Back Strap**.

NOTE: Position the **Upholstered Back** so that the “wider side” is closest to **Upholstered Seat**. Threaded bolt (on hinge) should protrude through the slot in **Back Strap**.

10. Install (1) nylon washer over the threaded bolt (on hinge), then install the **Back Height Adjusting Knob** and tighten to snug.

Operation of Stool

11. The stool is now ready for use. To raise the stool seating height; lift **Release Handle** without weight on seat.
12. To lower stool seating height; Lift the **Release Handle** while sitting on the seat; Release the handle when desired height is reached.
13. Position the **Upholstered Back** and **Foot Ring** into the desired positions using the **Adjusting Knobs**.

Important - To ensure proper operation of your Stool; periodically re-check tightness of all fasteners.

Wrinkles in Vinyl – Storage and shipping can cause wrinkles of the vinyl on your stool. This is normal. Please allow 1-2 weeks at room temperature (approx. 70 degrees) to allow for vinyl to expand back to original appearance.



Periodically recheck tightness of all fasteners, and never stand on the Upholstered Seat.