

V2 Series Treatment Table

Hamiltomedical
built for life

CUSTOMER INSTRUCTIONS

PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE USING THIS PRODUCT. If you have trouble understanding these instructions contact your dealer or Hamilton customer support, (888) 871-5600 before attempting to use this product; otherwise injury may occur.

Hamilton Medical assumes no responsibility for damage or injury caused by improper assembly, installation, use, or maintenance of these products.



1. READ AND FOLLOW ALL DIRECTIONS.
2. NEVER stand on table or INJURY MAY OCCUR.
3. NEVER allow user to sleep unattended on table or INJURY MAY OCCUR.
4. NEVER leave an unconscious user unattended on table or INJURY MAY OCCUR.
5. DO NOT use table for transporting user in or with ANY type of vehicle or trailer. The tables have not been tested or approved for use by an occupant in any type of vehicle or trailer.
6. NEVER leave an infant or child unattended on table or INJURY MAY OCCUR.
7. DO NOT put hands, feet or clothing into any openings when changing positions on table. Attendant MUST confirm that users hands and feet are safe while changing table positions or INJURY MAY RESULT.
8. Periodically, recheck tightness of all nuts, bolts and screws.
9. USE ONLY AUTHORIZED REPLACEMENT PARTS.
10. NEVER EXCEED the recommended **weight capacity of 400 lbs.** (181.4 kg).
11. SAVE THESE INSTRUCTIONS for future reference and training.

V2 Series Weight Capacity = 400 lbs. (181.4 kg.)

Tools needed:

- Phillips-head screwdriver
- 7/16" wrench (multiple size wrench included)
- Scissors or box cutter
- Cardboard shipping container (stays under table during assembly to protect upholstery)
- - If using powered tools always start hardware together by hand first & wear safety goggles

PLEASE READ AND FAMILIARIZE YOURSELF WITH ALL INSTRUCTIONS BEFORE PROCEEDING WITH ASSEMBLY

1. After checking your product for any shipping damage, table needs to be positioned upside-down on cardboard or protective surface. Remove all packing material and hardware that was secured for shipping. Cut tape on corners of box so that cardboard will lay flat while you assemble table.
2. You will find two **Welded Leg Assemblies**, in-between and attached to the **Table Frame Rails**. Pivot up the (2) **Welded Leg Assemblies**, (see sketch).

NOTE: The leg assemblies have braces that need to align with two threaded holes in the wood of the **Upholstered Top**.

3. Install (4) ¼-20 x 1-½" screws through (2) holes in each cross brace of the **Welded Leg Assemblies**, and into (2) threaded holes (each end) in the wood of the **Upholstered Top**, as shown in sketch.

NOTE: Do not fully tighten at this time.

4. Place (4) ¼-20 x 1-½” screws into the (4) holes through the **Lower Leg Brace** of the **Welded Leg Assemblies**, as shown. Heads of screws should face the ends of table, for better appearance.
5. Align the **Welded Brace Assembly** (so that it will be smooth-side-up when table is upright) with the (4) screws that you just installed on step 4, and fasten with (4) ¼-20 nuts.

*If the **Optional Shelf** and **Reinforced Welded Brace Assembly** have been ordered; make sure that the smooth surface (side with tape) of the **Reinforced Welded Brace Assembly** will face upward (when the table is in the upright position) prior to fastening to **Welded Leg Assemblies**.*

6. **TIGHTEN ALL FASTENERS AT THIS TIME** (including the nuts & screws holding the **Welded Leg Assemblies** to the **Table Frame Rails** – see sketch).
7. Install the four (4) black plastic **Threaded Inserts** into the ends of the table legs and screw the four (4) **Glide Feet** completely into the **Threaded Inserts**.

Additional instructions for other models ----- (V2 Series go to step 11)

8. Attach the **Face Slot Tube** so that one hole is aligned with the center hole of the **Upper Brace**, (see photo, flat side of Face Slot Tube faces table bottom). Position the opposite end of the **Face Slot Tube** over the 1” nylon spacer and the hole in table bottom. Install two (2) 1-3/4” Panhead Phillips screws through both ends of the **Face Slot Tube**, then tighten into the wood table bottom with a Phillips Head Screwdriver.
9. Install **Face Paper Holder** using (2) 1” “drywall-type” screws in pre-drilled holes on either side of Face Slot.
10. Install **Face Paper Cutter** on **Table Frame Rail** (where desired) opposite **Face Paper Holder**. Peel backing from tape on back of cutter, press cutter firmly into place.

*After peeling paper from adhesive tape:
Use damp sponge to moisten tape to allow face paper cutter to be adjusted during install.
Once tape is dry face paper cutter will stay in place.*

11. Using at least two people, turn table upright.
12. Place **Paper Holder Rod** through the (2) **Paper Rod Clamps**, (with paper roll in-between).
13. Adjust (4) 5/16” **Adjustable Glides** to level table.

Optional Shelf installation:

14. Double-sided tape has been installed on the Welded Brace Assembly.
15. Before removing paper on tape, “test fit” the shelf. Shelf should be approximately ¼” from edges of Lower Leg Brace.
16. Once you are certain of where you will position the shelf, peel off the paper to expose adhesive.

*After peeling paper from adhesive tape:
Use damp sponge to moisten tape to allow shelf to be adjusted during install.
Once tape is dry shelf will stay in place.*

- Carefully position shelf in place on adhesive.

Vinyl Strap Adjustment – Should the vinyl strap on your table need to be adjusted, unscrew the screw that currently holds either side of it in place. Pierce another hole in the strap to give desired tension and re-insert screw.

Wrinkles in Vinyl – Storage and shipping can cause wrinkles of the vinyl on your table. This is normal. Please allow 1-2 weeks at room temperature (approx. 70 degrees) to allow for vinyl to expand back to original appearance.



Do not over tighten nuts and bolts. Periodically recheck all fasteners for tightness.