

Operation Manual

Versatility Treatment Series

Model 6K8 Adjustable Back Table

Model 6K9 Flat Table

Model 6K10 Adjustable Back Extended Table

Model 6K12 Flat Extended Table

Hamiltomedical

1700 W. St. Paul Avenue - Milwaukee, WI 53233

P. 800/290-5960 - F. 414/342-4239

Foreword

You're the owner of the finest manual treatment table available in the market today. You'll find that your Versatility is a steady, strong companion offering years of service.

Please read this manual thoroughly so that you, the operator, have an understanding of how the table performs before using it with your first patient.

This treatment table has been designed for patient comfort, as well as for operator convenience and efficiency. We hope you enjoy using your new table. Please feel free to access our website for information on our other products: www.hamiltonmed.com.

HAMILTON MEDICAL FURNITURE CORPORATION

© 2012 Hamilton Medical

All rights reserved. This material may not be reproduced or copied, in whole or in part, without the written permission of Hamilton Medical Furniture Corp.

Table of Contents

SECTION		PAGE
1	GENERAL OVERVIEW	
	Scope	3
	Introduction	3
2	PRODUCT	
	Specifications	3
	Upholstery Fire Code	3
	Options	3
	Features	4
3	OPERATING INSTRUCTIONS	
	Warning	5
	Table Adjustments	5
4	TABLE CARE	
	Cleaning	5
	Upholstery	5
	Inspection Tips	5
5	WARRANTY	
	Limited Warranty	6



General Overview

Scope:

The purpose of this Operation's Manual is to familiarize you with the function, features, care and specification of your Versatility Manual Treatment Table. Included is information on the Versatility Series of Treatment Tables; 6K8, 6K9, 6K10 and 6K12

The Versatility Series of treatment tables have been designed with extra deep upholstery to provide the best in patient comfort while treatment is underway. Hamilton uses only the finest approved healthcare vinyl which is easy to keep clean and because it's UV stable it will remain color safe for years to come.

The 30" table height is not only the desired height physicians prefer, but is the perfect musculoskeletal height designed for the physicians ergonomic well being.

Introduction:

Table steadiness is the most important requirement in a treatment table. Our cross base, reinforced steel leg design gives Versatility that solid strength that physicians look for.

At the head of each Versatility table is a Ster-O-Sheet™ paper roll dispenser for an extra measure of sanitary protection.

If floors are even slightly uneven, the adjustable feet, two per leg, stabilize the table.

The 6K8 Versatility offers an adjustable table top with infinite number of positions. The pneumatic gas tube allows the back to be changed, is controlled by levers on each side of the table.

Product

Model Number:

6K8 Adjustable Top
6K9 Flat Top
6K10 Extended Adjustable Top
6K12 Extended Flat Top

Table Weight:

6K8 - 115 lbs. - kg.
6K9 - 170 lbs. - kg.
6K10 - 175 lbs. - kg.
6K12 - 120 lbs. - kg.

Specifications:

Height: 30 in. - 76.2 cm.
Width: 29 in. - 73.7 cm.
Length: 72 in. - 182.9 cm.
Extended Length: 78 in. - 198.1 cm.
Backrest Length - 6K8: 33 in. - 83.83 cm.

Patient Load: 500 lbs. - 226.8 kg.

Upholstery: Seamless healthcare vinyl, UV stable, stain resistant; over high density foam; upholstered to 3/4" plywood substrate.

Steel Frame Finish: Chemical resistant epoxy powder coat - chameleon color

Fire Code:

Fire code ratings are based on the State of California guidelines.

Versatility upholstery complies with CAL 117

Versatility has optional upholstery sets that comply with CAL 133

Options:

Model : 6K411 Two Drawer Accessory Kit, all steel frame, can be field installed or ships assembled to table. Seamless polystyrene drawers measure 19.25 in. x 17 in. x 4 in. (49.2 x 43.2 x 10.2 cm.)



Product Continued

Features:

Seamless Healthcare Vinyl is stain resistant, UV stable, easy to clean and made with thick, high density foam.

Ster-O-Sheet™
Paper Roll Dispenser

Chameleon epoxy
powder coated finish

Screw type rubber
padded feet levelers.

6K9

Outrigger design with cross
base reinforced steel legs offer
great stability.

Pneumatic Gas Tube provides one-
touch infinite back adjusting.

Option: 2 Drawer Kit
fits any Versatility model
and can be field installed.



Operating Instructions

Care should be taken to make sure that when the back of the Versatility is being adjusted that all parts of the operator's body and the patient's body are clear of the table's moving parts. Failure to do so could result in personal injury.

Warning:

Standing on the table for any reason is not recommended and by doing so could result in personal injury.

Using the Versatility for any other purpose other than for treating patients is not recommended. The upholstery is designed for treatment of people, not animals.

Adjustments:

The Versatility top on the 6K8 is fully adjustable from a supine position to 90°. To raise the back simply pull upward on the front of the table top. To lower the top use the release levers which are found on both sides of the table. Pulling the release lever upward allows the top to lower to the desired position.

The Ster-O-Sheet™ paper dispenser is located at the head end of the table. Slide the roll of paper (not provided) onto the wooden dowel which is easily released from its brackets by depressing the metal prong holding it in place. Once in place the roll of protective paper rotates on the dowel. When dispensing, pull paper up and over the full length of the upholstery.

If the table is not sitting on the floor properly and if it wobbles, the four feet at the bottom of each end of the horizontal portion of the leg can be adjusted. The feet are screw type and can be screwed in or out to achieve a level position for the table.

Table Care

Cleaning:

Painted surfaces and laminated drawer panels should be cleaned weekly with a standard soap and water solution.

Upholstery should be cleaned daily with a 5% Bleach and water solution. A specialized vinyl cleaner available through most custodial supply distributors may be used on the vinyl if desired.

Hamilton suggests that using medical wipes which contain alcohol to clean the upholstery may cause the upholstery to crack prematurely.

After cleaning with either water solution, we recommend that the surface should be wiped with a water damped cloth or sponge.

Inspection:

At least once a week inspect the table for any loose screws or bolts. If found, tighten them immediately.

Check for any missing pieces, or damaged components. Replacement parts may be obtained by contacting your Hamilton Customer Service Representative at 800/290-5960.

Warranty

WARRANTY – Hamilton Medical Furniture Corporation (HMFC) warrants its products purchased hereunder to be free of defects in materials and workmanship for a period of one (1) year from the date of shipment. A copy of the invoice which has the date of shipment establishes proof of warranty.

This warranty shall not apply in the event products are damaged as a result of misuse, abuse, neglect, accident, improper application, modification or repair by persons not authorized by HMFC, where products are damaged during shipment or where the date stamp on the product has been defaced, modified or removed.

All implied warranties including, but not limited to, warranties of merchantability and fitness for a particular purpose are hereby excluded.

This warranty is on parts only, not labor. Any labor warranty is at the dealer's discretion.

Purchaser's remedy, if any, for any defective product shall be limited to a refund, replacement or adjustment by HMFC of the parts required at HMFC's option, and shall in no event include damages of any kind, whether incidental, consequential or otherwise.

Parts shipped as warranty will be invoiced at their cost. A credit to our customer's account will be issued when the defective parts have been returned with serial number and purchase date verification.

Hamilton reserves the right to make any changes in design which, in our opinion, will improve the appearance, general construction, as well as performance, of our products. We also reserve the right to make substitution of critical materials if shortages develop.

Hamiltomedical